


How to Make a Shot-Glass Style E-Collar

Illustrated Directions by Jen Bailey



Step 1: Trace template onto clear plastic notebook cover, then cut out.

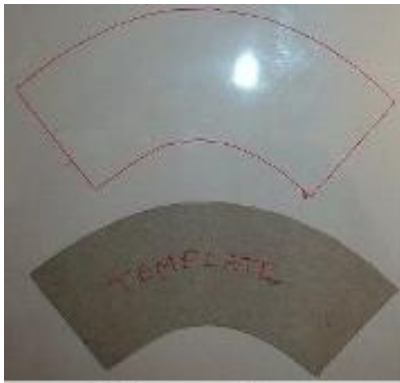

Step 2: Using 1 sheet of Dr. Scholl's Molefoam, cut 2 strips $\frac{1}{4}$ " along edge.



Step 3: Remove 1 adhesive strip and adhere one strip of molefoam to the other, retaining 1 adhesive strip at bottom for later use.




Step 4: Cut $\frac{3}{4}$ " off end of dual layer Molefoam. Remove adhesive backing and adhere to clear plastic as shown.




Step 5: Using 1 sheet of Dr. Scholl's Moleskin, cut 1 strip $1\text{-}\frac{3}{8}$ " wide down length.



Step 6: Cut $\frac{1}{2}$ " deep slits at $\frac{1}{2}$ " intervals down length of Moleskin strip.

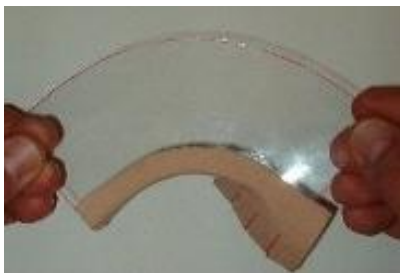

Step 7: Turn collar over, Molefoam side down. Remove backing from Moleskin. Affix $\frac{1}{4}$ " of uncut edge to collar along inside edge.

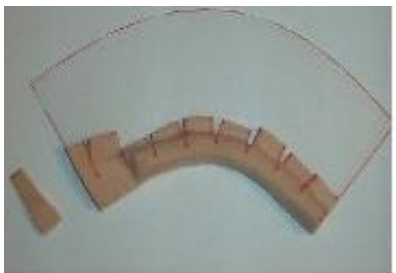

Step 8: From the middle of Moleskin, wrap around over Molefoam, maintaining contours of collar. Trim excess Moleskin from end, as shown.



Step 9: Cut $\frac{1}{2}$ " strip from Moleskin.



Step 10: Remove adhesive backing and adhere over slitted Moleskin, above thick layer, to cover the splits made from the contouring.



Step 11: Cut 1 strip of Moleskin 1-3/4" long by 5/8" wide.



Step 12: Remove backing and place 1/4" along edge of collar where Molefoam is thick, leaving an overhang of Moleskin.



Step 13: Wrap overhang of Moleskin around edge of collar, completely covering edge.



Step 14: Apply a 1.5" strip of duct tape. Fold 1/4" of duct tape over to make a tab for easy removal.



Step 15: Your e-collar is complete and ready to be placed on your sugar glider.

Scroll down for a printable full-size template.

Shotglass E-Collar Full-Size Template

Mole skin

Step 9

2x Mole foam

Step 2

Mole Skin

Step 10

Mole skin

Step 5-6

Template (full size)

